

**Actual Size:** 32" x 40" x 40" • (815.34cm x 101.6cm x 101.6cm)

**Shipping Size:** 35" x 43" x 43" • (88.9cm x 109.22cm x 109.22cm)

**Net Weight:** 200lbs • (90.8kg)

**Shipping Weight:** 280lbs • (127kg)

**Seat Width:** 25" • (63.5cm)

**Unoccupied Water Capacity:** 70gal • (265L)

**Occupied Water Capacity:** 25-55gal • (94.6 - 208.2L)

## Included Features (Soaking)

1. Acrylic shell gloss white finish fiberglass gelcoat reinforced
2. Stainless steel frame with adjustable leveling legs
3. Plastic composite and tempered glass outward swing door with 3 latch gear & shaft mechanism
4. Two (2) Access Side Panels
5. Single 2" 2-port PVC drains with 1 end caps included, 2 overflows, 2 stainless steel operated openers with extended handles in chrome finish

## Jetting & Therapy Options

1. Air + Hydro (LED Chromatherapy Light Not Included)
3. Hydro Massage + Microbubble (LED Chromatherapy Light Not Included)
4. Hydro Massage (LED Chromatherapy Light Not Included)
5. Soaking



## Jetting & Therapy Systems

### Hydro Massage

- Hydro Massage Pump 7A
- Intensity Dial Air Control
- Ozone Sterilization 0.04A
- 13 jets (11 Seat Jets 2 Calf Jets)
- On/Off Push Control
- In-line Water Heater 6A 650W

### Microbubble Therapy

- 1 HP Pump 7.3A
- Pressure Vessel
- On/Off push control
- Agitator Valve

### 2 Piece Faucet

- Up to 16GPM @ 60PSI (at ideal in-home plumbing conditions)
- Two 5' Water Supply Lines 3/4"
- Pull out Hand Shower in Chrome Finish with a 5' Retractable Stainless Steel Braided Hose

### Air Massage

- Air Massage Pump 4.5A 500W
- 3 Speed Pump
- Auto-purge (Activates 20 minutes after pump shutdown)
- 16 jets (10 Seat Jets 6 Foot Jets)
- On/Off push control

### Optional Features

1. Heated Seat and Backrest
2. Digital Controller

### Electrical Requirements



Dedicated GFCI protected electrical line(s) required 110/120V 60Hz

**Disclaimer\*:** AMP value for each pump may change based on availability. Please verify before purchase. (800.480.6850)



### Warranty

- Lifetime limited\* warranty on shell, frame, door, and parts to original purchaser/Address
- Labor or installation is not covered - please refer to the Ella warranty certificate
- Non Ella brand faucets covered under manufacturers original warranty

# Front Entry Acrylic Outward Swing Door Walk-In Bathtub Specifications

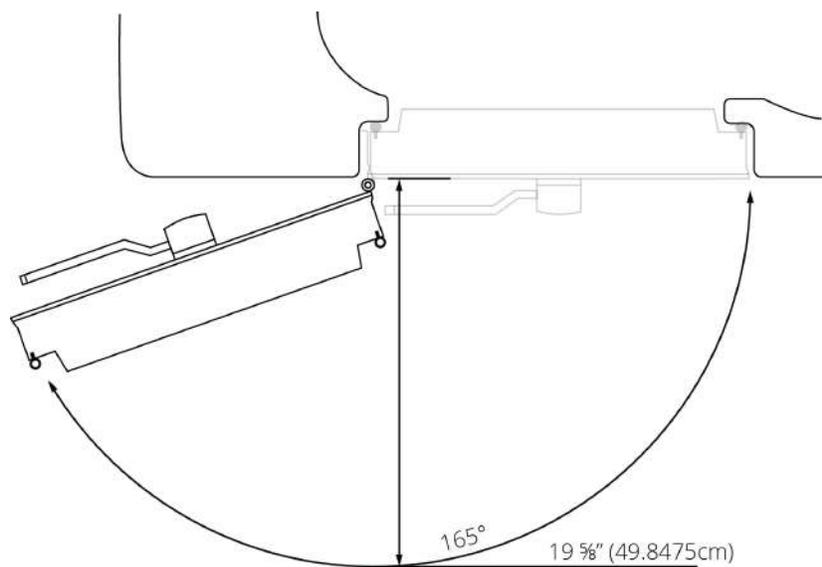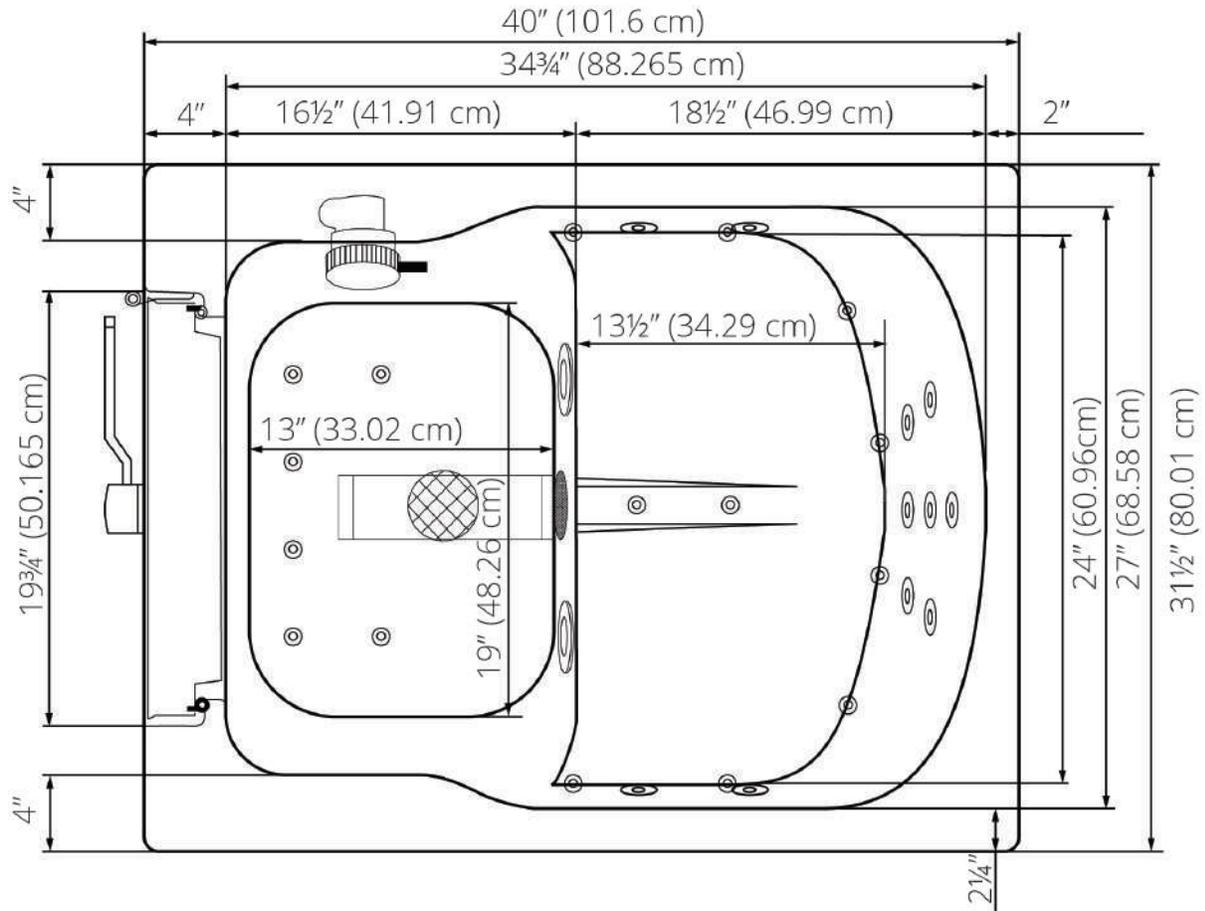


# Front Entry Acrylic Outward Swing Door Walk-In Bathtub Specifications

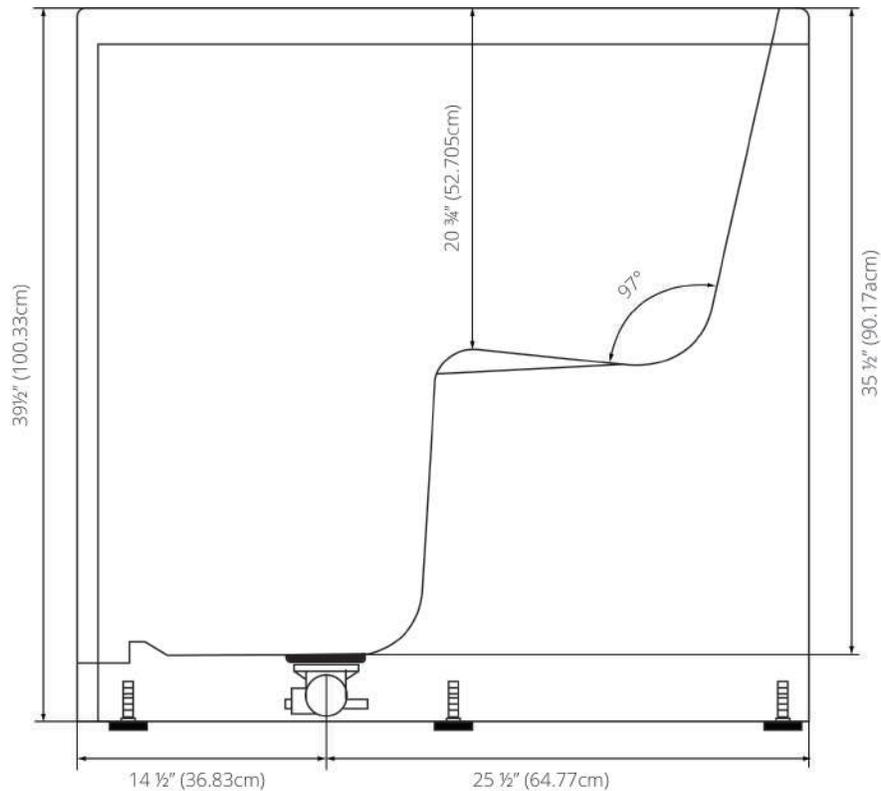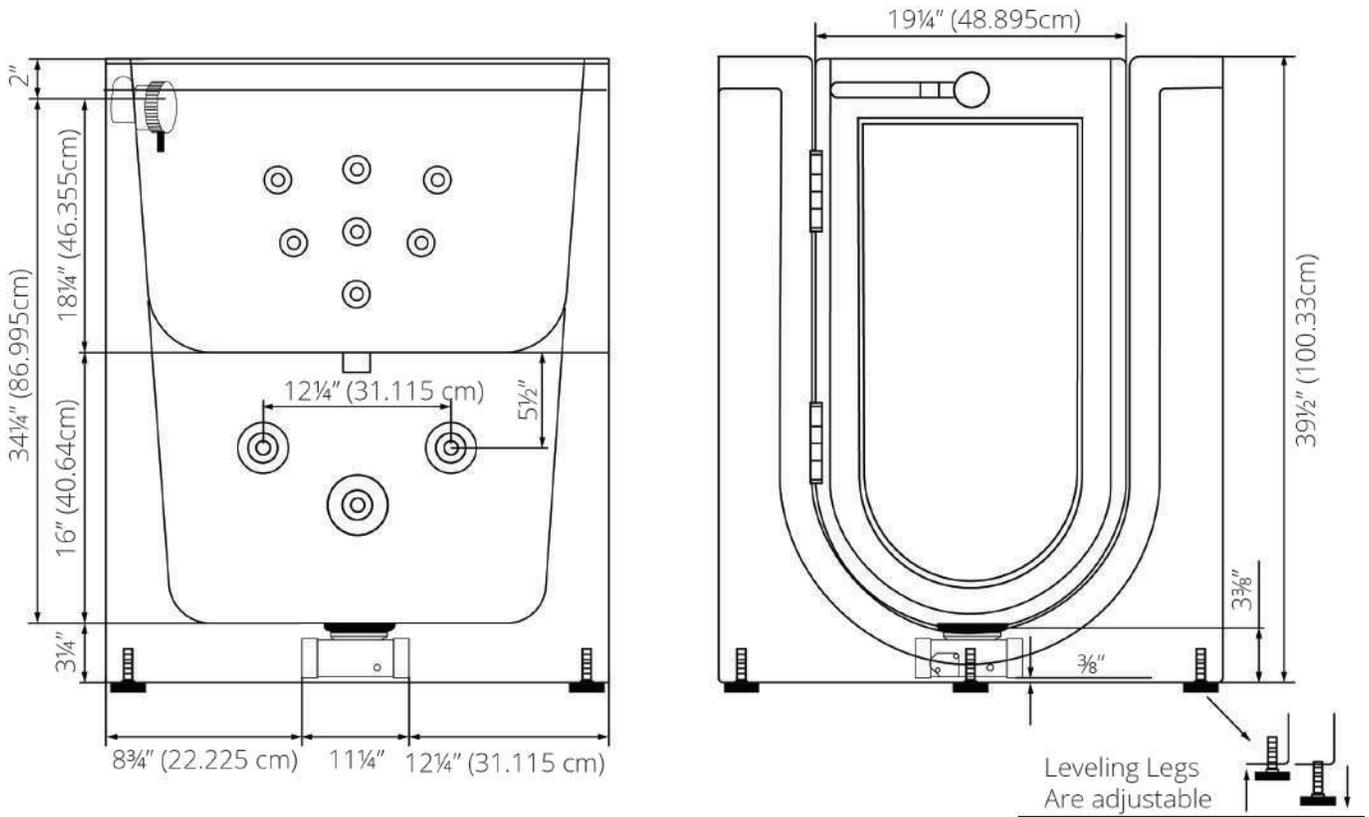

# Ella 2" T-Type Single Port Floor Drain (One Drain Included)

2" T-type Dual Port Floor Drain for  
Ella Dual Drain Walk-In Bathtubs

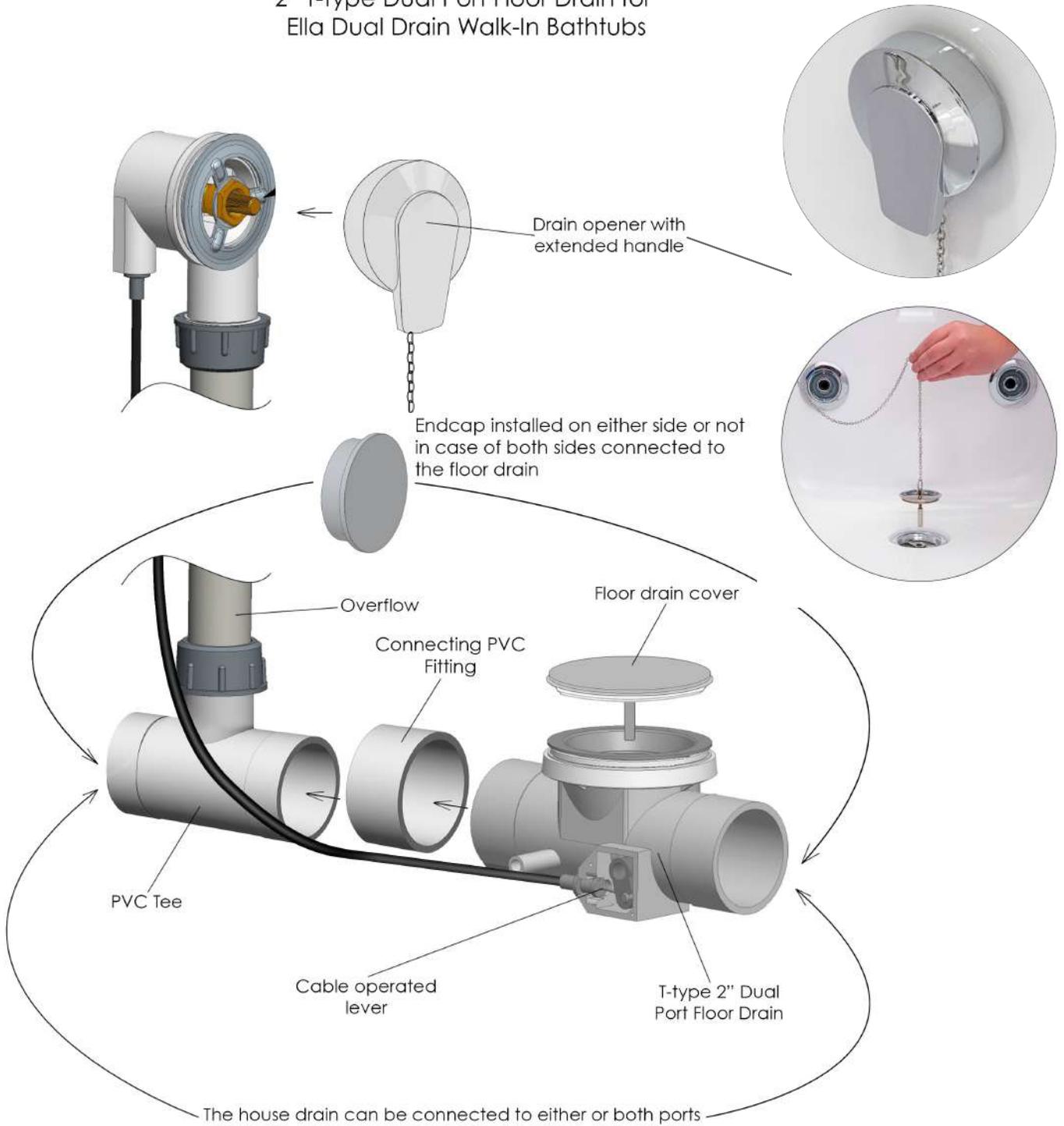

Two 3/4" connector water supply lines included with Ella Fast Fill Faucets  
Connect to Shut Off Valves (not included)




All electrical fixtures connect to a dedicated GFCI protected outlet or GFCI protected Circuit Breaker.



Images below show examples of PVC pipe connections (not included). Reducers can be used to connect smaller drain pipes.

T-Type four (4) 2" drain ports allow for the drain pipe to be installed in either direction.


Drain Caps Installed On Either Side Of The Drain unused Port.

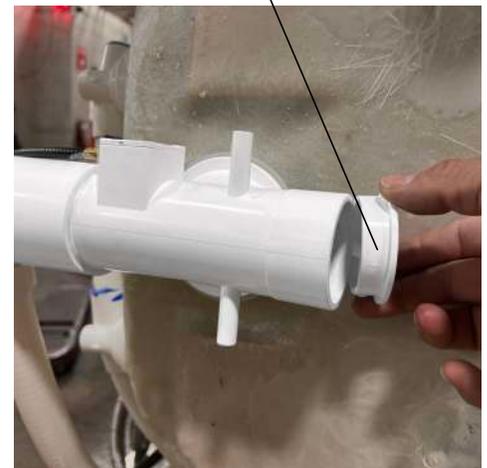



Step-by-step Instructions to replace a valve cartridge for Ella 2-piece faucets.


Step.1 Remove the Hot/Cold screw hole cover.

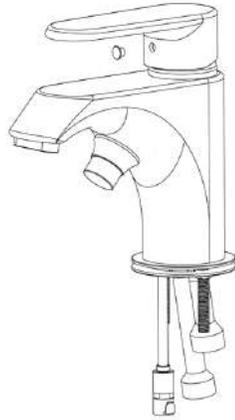


Step.2 Use an Allen wrench to remove the handle Screw.



Step.3 Take out the handle and remove the brass escutcheon

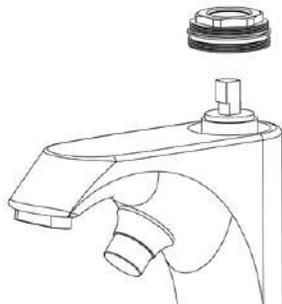

Step.4 Use a wrench to unscrew the pressure cap.

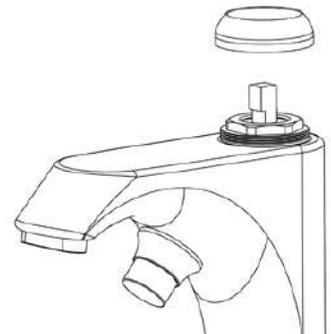

Step.5 Remove the cartridge and replace it with new piece.

